

## 7 Signals during operation of the stairlift



The lamp on the armrest control indicates the status of the stairlift.

Signal	Meaning
Constant green lamp.	The joystick has been operated and the stairlift is moving or will move.
Constant green lamp + audio signal from motor.	After the stairlift has been operated, the stairlift is operated again with a different control. The stairlift will start moving within 10 seconds.
Flashing green lamp.	The batteries of this control are (almost) empty and must be replaced.
Flashing red lamp + audio signal from motor.	1. The stairlift is not being charged. Drive the stairlift to a charging contact. <b>Or:</b> 2. The stairlift has encountered an obstacle. Drive the stairlift in the opposite direction and remove the obstacle.
Constant red lamp.	There is a malfunction in the stairlift. Switch the stairlift off and then on again. If the stairlift does not work, contact your supplier.

## 8 Replacing the batteries of the armrest control



Remove the cover by placing the joystick in the cutout under the cover and then turning it a half turn.



Replace the batteries with the spare batteries located in the other armrest.



Refit the cover.

# Flow stairlift Quick start manual



This is a quick start manual. You should read the full user manual, which includes important advice and instructions.

## Unfolding the seat and sitting down

1



Unfold the seat.



Sit down.



Fold out the footrest.



Fasten the safety belt.



Fold down the armrests.

## 2 Operating the stairlift



Insert the joystick.



Hold the joystick in the required direction for the entire journey.



Attendant control (optional).

- There is a short delay between pressing the joystick and the lift moving. You will hear an audio signal indicating that the stairlift is about to move.
- The stairlift stops automatically at the end.
- Always park the stairlift on a charging contact, to prevent the batteries from becoming empty.
- If you do not park the stairlift on a charging contact, the stairlift will emit a continuous audio signal.

## 3 Standing up and folding up the seat



If desired, remove the joystick by pushing it forward.



Fold up the armrests.



Release the safety belt.



Fold up the footrest.



Stand up.



Fold up the seat.

## Using the optional manual swivel seat

4



Location of swivel lever.



Swivel the chair by pulling the lever upwards.



Lock the swivel seat by pushing the lever firmly downwards.

## Using the call and park stations

5



Insert the joystick.



Calling the stairlift.



Parking the stairlift.

- Hold the joystick during the entire journey.
- Always park the stairlift on a charging contact.
- If you have an automatic swivel seat (optional), the seat will automatically turn in the required direction before sitting down or sending it to a charging contact.

## Main switch

6



**Push in:** The stairlift is switched off.

**Pull out:** The stairlift is switched on.



An illuminated lamp on the drive unit cover indicates that the stairlift is switched on.



If a malfunction occurs, you can let the stairlift descend, by pulling out the main switch.